



**D-Race 1**  
30/.10

800-1000RPM – 1.5Ncm

**XP-endo® Rise Shaper**  
30/.04

1000-2500RPM – 1Ncm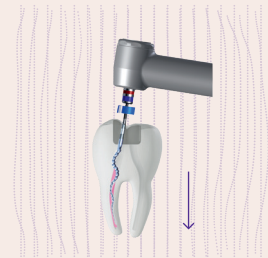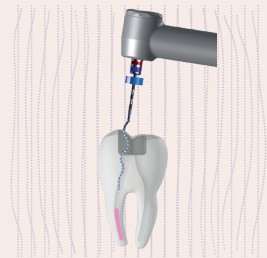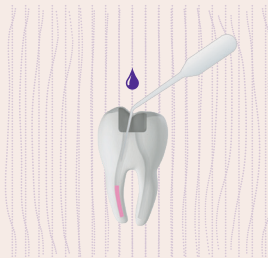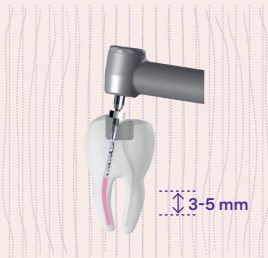

### EN

- Create a start in the coronal portion of the gutta percha (3-5mm) using D-Race 1.

- Place one drop of solvent in the coronal space created and wait for at least 1 minute.

- Place the tip of XP-endo Rise Shaper into the prepared space in the gutta percha, disengage slightly and start the motor at 1000-2500 rpm.

- Use pecking motion until the XP-endo Rise Shaper's tip engages the gutta-percha. Then use light pressure to help advance the XP-endo Rise Shaper down the canal until working length (WL).

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



800-1000RPM – 1Ncm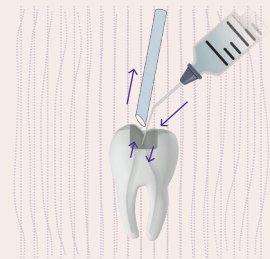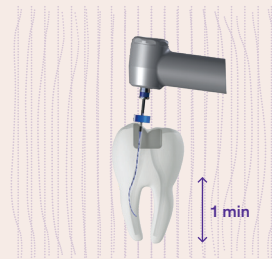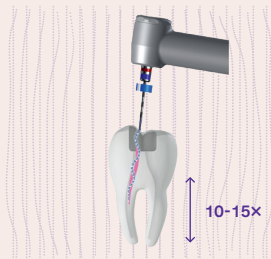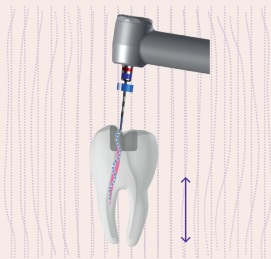

**EN**

• Allow the XP-endo Rise Shaper to gently corkscrew around the gutta percha and tease out large strands of gutta percha using long gentle strokes.

• After the removal of the gutta percha, measure the WL, irrigate and fill the pulp chamber and canals with NaOCl. Use the XP-endo Rise Shaper for 10-15 additional long gentle strokes to WL. Recommended speed: 1000 rpm

• Use the Finisher R for at least 1 minute (longer, if debris is still coming out of the canal), using slow and gentle longitudinal movements to contact the full length of the canal.

• Use copious NaOCl irrigation and suction to remove the gutta percha and sealer tags prior to complete the treatment.

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